
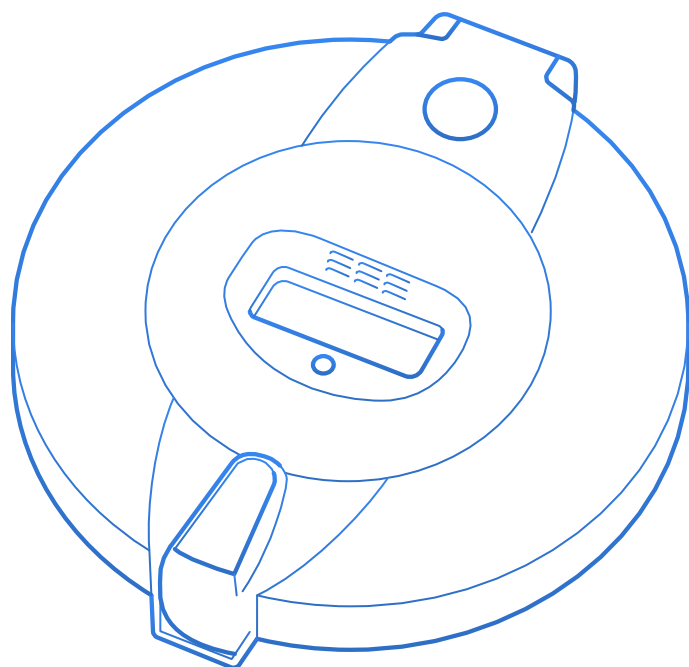
Pill Dispenser

User Manuals

NE41 22007-00 v2.1

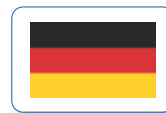[English](#)

[Deutsch](#)

[Español](#)

[Svenska](#)

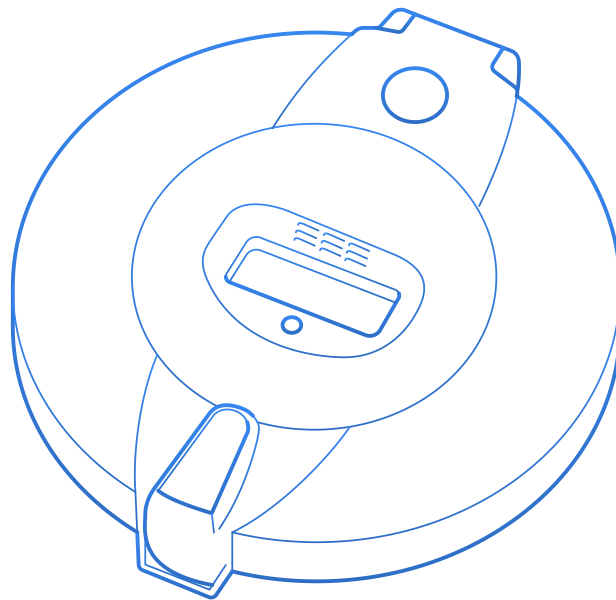
[Français](#)

Pill Dispenser

User Manual

NE41 22007-02 v2.1

Declaration of Conformity: Hereby Legrand AB declares that this device is in compliance with Directive 2014/53/EU. The full text of the EU Declaration of Conformity is available at:

www.legrandcare.com/declarations-of-conformity

Table of Contents

1 About Pill Dispenser	5
1.1 Contents in the Box	5
2 Your Pill Dispenser	6
2.1 The SECURE mode	6
2.2 The Display	7
3 Get Started with Pill Dispenser	8
3.1 Settings	9
3.1.1 Menu A	9
3.1.2 Menu B	10
4 How to fill the dispenser	14
4.1 Re-set the DOSES LEFT Display	15
4.2 Lock	15
4.3 Battery Cover Screw	16
5 General Information	17
5.1 Self Test Function	17
5.2 Dispenser Left Upside Down	17
5.3 Error Alarm	17
5.4 Dose not Taken Alarm	18
5.5 Low Battery Condition Alarm	18
5.6 Out of Range Alarm	18
5.7 Alarm Test Call	19
6 Connect the Pill Dispenser to the Telecare System	20
6.1 Register the Pill Dispenser in Pulse CMP	20
6.2 Local Pairing of the Pill Dispenser	20
7 Safety and Handling	21
7.1 Cleaning Pill Dispenser	21
7.2 Discard	21
8 Technical Data	22
Appendix A Connect Pill Dispenser to Telecare Receivers	23
1 Register the Pill Dispenser on a Reach IP	23
2 Pair the Pill Dispenser with Carer Response	24

3	Register the device in Touchsafe Pro	25
4	Register the device in Advent XT2	26
5	Register the Pill Dispenser on a NOVO	27
5.1	Through Pulse CMP	27
5.2	Through the Service Mode in the NOVO	28

1

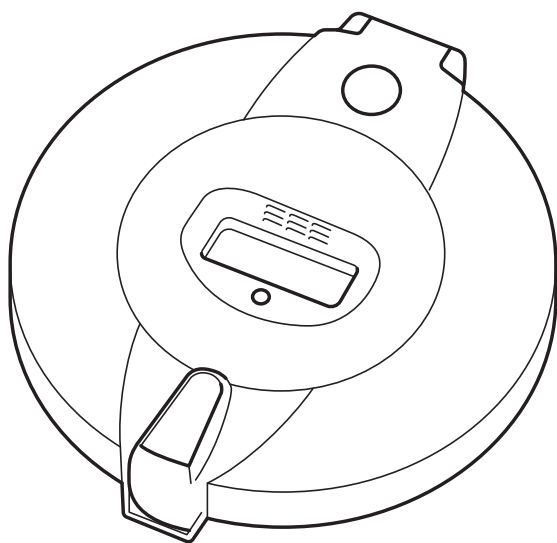
About Pill Dispenser

The automatic Pill Dispenser reminds the user with an alarm and a flashing light when it is time to take medicine. It dispenses the correct dose at the correct time during the day or night, and all other pills stay out of reach. The medication is dispensed from a tray with 28 compartments inside the device. A maximum of 24 alarms can be set for each day.

The Pill Dispenser can be placed on any flat surface in the home.

Compatible with the Legrand Care telecare system, including surveillance of the device's status and notification to the alarm central if the user has not taken their medicine.


1.1 Contents in the Box

- Pill dispenser
- 4 x AA alkaline batteries
- 3 reversible day/time discs
- 1 key for locking the dispenser
- 1 battery cover securing screw
- Pharmacy label carrier + clip

2

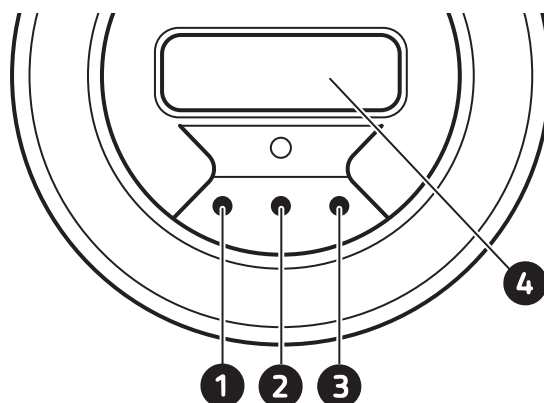Your Pill Dispenser

- 1 Button 1
- 2 Button 2
- 3 Button 3
- 4 Display

2.1 The SECURE mode

When operating in 'SECURE' mode a self-locking shutter over the lid opening prevents access to the medication tray except at the time that medication is programmed to be available. This prevents the possibility of the user double dosing or tampering with the medication tray.

When the dispenser lid is closed the shutter locks making the medication tray inaccessible. At the dosage time the tray will advance clockwise a half section releasing the shutter. The alarm sounds and the light flashes to alert the user.

The medication can be accessed by turning the dispenser over and pouring the pills into the hand or suitable container. The shutter will swing open as the dispenser is tilted allowing the medication to fall out. When the dispenser 'beeps' (at approx. 135°) the alarm will be cancelled and the dispenser can be returned to the upright position.

When operating in 'SECURE' mode the shutter in the lid of the dispenser is always locked except at the dosage time and subsequent 'AFTER' time (i.e. the time set for the alarm to continue).

The dispenser detects when the medication has been dispensed and locks the shutter 2 minutes after dispensing by rotating a further half section thereby preventing access to the tray.

The shutter will also lock if medication is not dispensed within the programmed alarm time set by the 'AFTER' time. This time can be set from 5 minutes to 5 hours. This function eliminates the possibility of the user taking two doses within a short time and thereby potentially double-dosing.

**NOTE**

The user has the option of switching the SECURE function 'OFF' if preferred.

2.2 The Display

The LCD display is visible through the lid. The display shows the following:

- the date and time
- the time for the next medicine dose
- the number of doses that remain
- the battery level

When the dispenser is horizontal the display shows the Date and the Time in sequence.

At the alarm time '**TAKE MED**' is displayed.

When tilted forward slightly (between 15 and 40 degrees) the display will show in sequence:

- **NEXT DOSE 16:30 MON || DOSES LEFT 9 || B (battery level) 75% || DATE: 26MAR2024** (the values shown above are examples only).

When only 4 doses remain the horizontal display will additionally show **DOS 04**. After this dose has been dispensed **DOS 03**, **DOS 02**, **DOS 01** and finally **DOS 00** will be displayed in sequence with the **DATE and CLOCK**.

When the battery capacity is less than 20% the horizontal display will show **LOWBATT** in sequence with the **DATE and CLOCK**.

This LCD display feature enables the user, carers and family members to make a visual check on the status of the dispenser without having to open it.

3

Get Started with Pill Dispenser

The pill dispenser is a free-standing unit which can be placed on any flat surface within the home.

- 1 Open the lid as in the illustration below: place an index finger in the lid opening and press in the catch with a thumb. Lift the lid.

Note: When opening for the first time, the lid is unlocked.

The key and the screw that secures the battery case is in one of the pill compartments.

- 2 Open the lid to the battery box beneath the dispenser and activate the dispenser by removing the battery isolating strip. (**Note the correct battery polarity + / -**)

- 3 The display shows "BOOT" followed by the software version e.g. "SW-007"; then the battery charge level "B 100%" (full charge). When the display shows "CLOCK" and flashes "00:00" the unit is ready for programming.

To avoid excessive battery drain it is important that the clock is set soon after either the isolating strip is removed, or the batteries are replaced.

IMPORTANT

Do an alarm test call when the Pill Dispenser is new, and within 9 hours after the batteries have been replaced.

3.1 Settings

The dispenser is set using the three buttons:

Button 1 is used for moving to the next function.

Buttons 2 and 3 are used to change the settings in each function.

There are two setting menus:

- **Menu A**

To access press Button 1

Functions: **CLOCK, DATE, DOSES, ALARMS**

- **Menu B**

To access press and hold Button 3 until 'BEFORE' is displayed

Functions: **BEFORE, AFTER, SOUND, LIGHT, FILLED, BATT, SECURE, HOURS, DLS, START, STOP, RECORD.**

After completion of settings in Menu A or B press button 1 and hold down for three seconds to restore to normal operation.

If there is no programming activity in Menu A the display will automatically return to normal after 25 seconds.

If there is no programming activity in Menu B the display will return to normal after 5 minutes.

3.1.1 Menu A

To access the functions in Menu A press and hold button 1 until 'CLOCK' is displayed on the screen.

NOTE

To ensure correct operation it is most important that both the 'Clock' and the 'Date' are set before the dispenser is used. When the dispenser is activated the display will prompt the setting of the Clock and then the Date

CLOCK	<p>Current time</p> <p>Press Button 1 until 'CLOCK' is visible on the display.</p> <p>Set the correct time by pressing Buttons 2 or 3. You can fast forward or reverse the time by pressing and holding down these buttons.</p> <p>–</p>
DATE	<p>Current date</p> <p>Press Button 1 again – 'DATE' is visible on the display.</p> <p>Wait a few seconds. YY will be displayed.</p>

Using buttons 2 and 3: Set the Year (YY) then press 1

Using buttons 2 and 3: Set the Month (MM) then press 1

Using buttons 2 and 3: Set the Date (DD) then press 1

-

DOSES **Number of set doses per day**

'DOSES' will be displayed.

Wait a few seconds and then adjust the number of doses per day using buttons 2 and 3.

Press button 1

-

ALARMS **Alarm times for the daily doses**

'ALARMS' will be displayed.

Set the 1st (1) alarm time using buttons 2 or 3.

Press Button 1 again for the 2nd (2) alarm time and repeat the procedure for all subsequent alarm times required (Maximum 24 daily).

-

Press and hold down button 1 for a few seconds to exit the menu and revert to normal operation.

3.1.2 **Menu B**

To access the functions in Menu B press and hold button 3 until 'BEFORE' is displayed on the screen. The individual functions can then be accessed by pressing button 1 repeatedly.

BEFORE **Taking medication before the pre-set alarm time**

It is possible to allow medication to be taken up to 5 hours before the normal pre-set alarm time. To set this function go to 'BEFORE'

Press Buttons 2 or 3 to set the time required.

If this function has been set the user may access their medication in advance by turning the dispenser upside down (through 180°) and then upright *twice* in succession. The dispenser will then rotate one step and the next dose will become available.

Note: If medication is taken before the normal time, the dispenser will not activate again until the following medication is due.

If this function is not required set at 00.00.

-

AFTER The length of time the alarm will sound / light will flash can be adjusted by the 'AFTER' function. This also controls the time that the shutter will remain unlocked.

Press Button 1 until 'AFTER' is displayed.

Select the required alarm duration by pressing Buttons 2 and 3 (minimum 5 minutes / maximum 5 hours).

The alarm will sound and light flash every 5 seconds for the first minute, every 8 seconds up to 1 hour and thereafter sound every minute and light every 16 seconds until either the medication is dispensed or the AFTER limit is reached, maximum 5 hours.

–

SOUND

There are 4 alarm sounds available. To choose the alarm sound:

Press Button 1 repeatedly until 'SOUND' is displayed.

Choose your preferred sound 01 02 03 or 04 by pressing Buttons 2 or 3.

Should no sound be required set at 00.

–

LIGHT

Press Button 1 repeatedly until 'LIGHT' is displayed.

If the light is not required press button 2 once – 'OFF' will be displayed.

If the light is required press button 3 once – 'ON' will be displayed.

–

FILLED

(The number of doses placed in the medication tray when filled)

This function enables the display warning 'DOSES LEFT'

Press button 1 until 'FILLED' is displayed.

Set the number of doses placed in the medication tray when filled using buttons 2 and 3.

When no doses remain the display will show '----'.

–

BATT

(Battery type)

It is possible to use both Alkaline and rechargeable NiMH batteries.

Press Button 1 repeatedly until 'BATT' is displayed.

Using buttons 2 and 3 set at either ALC for Alkaline and NIMH for rechargeable NIMH batteries.

–

SECURE

The dispenser can be set to operate in 2 modes:

Secure On: Press Button 2.

The shutter in the lid of the dispenser is always locked except during the programmed alarm time. The shutter will lock again 2 minutes after medication has been dispensed.

The shutter will also lock when the 'AFTER' time has expired, thus preventing the possibility of the user taking this dose a short time before the next dose is due.

–

Secure Off: Press Button 3.

When the dispenser is refilled with medication and is re-set the tray will move forward a half section, locking the shutter and thereby allowing secure transport of the medication to the user's home.

Note: After the first dose, the medication tray will always move one complete position at each alarm time leaving the shutter unlocked.

–

HOURS **24H / AM-PM clock settings**

The dispenser is pre-set to display the time with the 24 hour clock.

To change this to 12 hour AM/PM setting:

Press Button 1 repeatedly until 'HOURS' is displayed.

Choose AM/PM or 24hrs by pressing buttons 2 or 3.

–

DLS **(Daylight Saving)**

To enable the dispenser to automatically adjust for daylight saving (GMT/BST) press Button 1 until DLS is displayed.

Using Buttons 2 and 3 set at 'EU' for European Daylight Saving or 'US' for American Daylight Saving.

Set at 'OFF' to disable this function.

–

START DELAY

It is possible to delay the time at which the dispenser will start to operate. This function is useful if the dispenser is being filled, for example, by a pharmacy and is not being delivered to the user until a later date.

To activate this function press Button 1 until 'START' is displayed.

Set the Year (YY) Month (MM) Date (DD) and Hour (HH) that you wish the dispenser to activate following the same procedure as for setting the DATE (see Menu 1 above). The time (HH) should be in advance of the first dose to be dispensed for the user.

If the function is not required do not set.

–

STOP

It is possible to stop the dispenser's operation at a future date and time. This function is useful if two or more dispensers are being used by a user in sequence and ensures that only one dispenser is active at one time.

To activate this function press Button 1 until 'STOP' is displayed.

Set the Year (YY) Month (MM) Date (DD) and Hour (HH) that you wish the dispenser to stop operating following the same procedure as for setting the 'DATE' (see Menu 1 above). The time (HH) should be set after the last dose to be dispensed for the user.

If this function is not required set the year (YY) to 99.

RECORD

–
The dispenser will record the actual times the previous 28 medications were dispensed.

Press Button 1 until 'RECORD' is displayed – the time that the last medication was taken is displayed.

Press Buttons 2 or 3 for the previous medication times to be shown.

–

4

How to fill the dispenser

Note: Certain medication should not be exposed to the air and should therefore not be placed in automatic dispensers; if in doubt check with a Pharmacist.

1 Time discs

Three day / time discs are supplied:

Two printed discs for 1 up to 4 medication prompts per day (use either side) and one blank disc for more than 4 medication prompts per day.

The blank discs should be completed by hand indicating the medication times.

Place the appropriate disc for the dosage in question on top of the tray. Locating pins are provided on the medication tray with corresponding holes in the time discs for accurate positioning. (see **Fig. 3**).

Fig. 3

2 Medication

Remove the tray from the dispenser.

Place the correct dosage in each section of the tray.

Note: If a pill is inadvertently dropped in the incorrect compartment, a pair of tweezers can be helpful to easily remove the pill.

3 Replace the tray in the dispenser

After filling, place the tray in the dispenser positioned so that the compartment with the *first dose* to be taken by the user (in this example, Monday evening) is positioned to the right of the opening (see **Fig. 3**).

If the dispenser is set to operate in 'SECURE' mode, press Button 2 until 'TRAY Z' is displayed. The tray will then move forward one half section to the correct position.

Close the lid.

4 Pharmacy label carrier: (see Fig. 4).

- 1 Enter the medication details in the space provided on the pharmacy label carrier and, if appropriate, attach the pharmacy labels in the spaces provided.
- 2 Stick the label carrier to the battery case cover using the re-posable glue spot provided on the rear of the label carrier (first remove the glue spot cover).
- 3 Clip the retaining case over the pharmacy label carrier as shown.

Fig. 4

4.1 Re-set the DOSES LEFT Display

Each time medication is dispensed the number of 'DOSES LEFT' displayed will reduce by one (see [2.2 The Display](#)).

When the tray is re-filled it is important to reset the 'DOSES LEFT' to the actual number of doses placed in the medication tray (see [3.1.2 Menu B > FILLED](#)).

Replace the tray in the dispenser in the correct position (see [Fig. 3 in 4 How to fill the dispenser](#)) and press Button 2 until 'TRAY Z' is displayed. *If the dispenser is operating in SECURE mode the tray will move forward one half section.*

Close the lid.

The number of 'DOSES LEFT' displayed will now be the same as the number entered in 'FILLED'.

4.2 Lock

The dispenser can be locked with the key supplied. Turn the key a quarter of a turn clockwise to lock, anti-clockwise to unlock.

Please note that the lock is a simple device intended as an additional barrier only.

4.3 Battery Cover Screw

A screw is provided to prevent inappropriate access to the batteries (should this be necessary). Place the screw in the hole in the battery cover securing clip and tighten.

The screw must be removed when the batteries need replacing. It should then be replaced.

5

General Information

5.1 Self Test Function

Make a self test of the rotation, light and sound functions by pressing and holding all three buttons until TEST is displayed. Press button 1 to test the light.

Press button 3 to start a full alarm-sequence:

- 1 The display shows 'DEMO', the tray moves one half section to the open position, the Pill Dispenser makes a sound and the light flashes.
- 2 'TAKE MED' is displayed for 20 seconds.
- 3 Pick up the dispenser and turn it over to make a test of dispensing the medication. The display shows 'CLOSE' and the tray moves forward one half section to the closed position.
- 4 If the dispenser is not picked up, 'CLOSE' will be displayed after 20 seconds, and the tray will again move forward one half section to the closed position.

Press and hold button 1 to leave the TEST function.

5.2 Dispenser Left Upside Down

If the pill dispenser is upside down when an alarm occurs, the alarm will sound as normal, but the tray will not turn until the dispenser is returned upright. If more than one alarm time has passed whilst the dispenser has been upside down the dispenser will present the correct medication for that time.

5.3 Error Alarm

If the tray fails to rotate due to an obstruction an alarm will sound, and the display will show ERROR 01, 04 or 08.

This can be cleared by pressing button 3. Any obstruction should be removed. When the obstruction is cleared retest the unit. If the fault cannot be rectified contact your supplier.

5.4 Dose not Taken Alarm

The audible alarm can be set to sound for a minimum of 5 mins to up to 5 hours from the dosage time (known as the 'After' time). If a medicine dose has not been dispensed after half of the 'After' time, a 'Dose Not Taken' alarm is sent.

This gives the alarm center time to contact the user before the dispenser shutter locks at the end of the 'After' time. (see [2.1 The SECURE mode](#)) If the medicine dose still has not been dispensed after the full 'After' time, a second alarm is sent.

Pill Dispenser can send alarms regarding 3 different events: *Dose not Taken*, *Dose Ready* and *Dose Taken*. For information on which alarm types are supported by the receiver, contact your Legrand Care distributor.

5.5 Low Battery Condition Alarm

If the battery level is below 20% the display shows "LOWBATT" in sequence with the DATE/TIME.

If the batteries have not been replaced by 00:05:10 (5 minutes and 10 seconds after midnight) a low battery alarm is sent. This is repeated every night at 00:05:10 until the batteries have been replaced.

If the battery level is to 0% the pill dispenser will go OFF, it makes a sound, and the display shows "ERROR 10".

Check the battery level at any time by tilting the dispenser forward (between 15 and 40 degrees). After the "NEXT DOSE" and "DOSES LEFT" information the battery level is displayed, for example "B 75%".

Current programming is not affected by replacing the batteries. The time may need correcting if the batteries are left out or if they are completely out of charge.

5.6 Out of Range Alarm

The Legrand Care receiver can monitor all connected radio peripheral devices to determine whether they have been damaged or removed from an installation. If the receiver is unable to detect a device's heartbeat it will automatically generate a "radio out of range" alarm.

Firstly, investigate if there is a genuine reason for the alarm. If not, arrange for the device to be replaced immediately.

NOTE

Do an alarm test call within 9 hours of initial setup or after battery replacement to ensure digital heartbeats are transmitted.

5.7 Alarm Test Call

To check the alarm call function, do the following steps:

- 1 Open the lid on the Pill Dispenser.
- 2 Press and hold all 3 buttons until 'TEST' is displayed.
- 3 Press button 3. The display shows 'DEMO' and a test alarm is sent.

The light flashes and the display shows TAKE MED.

To exit the DEMO mode, press and hold button 1 on the Pill Dispenser.

6

Connect the Pill Dispenser to the Telecare System

6.1 Register the Pill Dispenser in Pulse CMP

The Pill Dispenser includes a radio transceiver to communicate with the Legrand Care receiver or carephone.

Login to Pulse CMP and enter the receiver's serial number in the Carephone view. Press the Accessories button, press *New* and enter the Radio ID code, Location and Equipment type. Press *Save* , then press *Synchronize*.

The Radio ID is a 10-digit hex code printed on a label on the Pill Dispenser.

6.2 Local Pairing of the Pill Dispenser

To do a local pairing of the Pill Dispenser with a Legrand Care receiver, follow the pairing instructions for the receiver in question. Make an alarm test call from the Pill Dispenser to confirm the pairing. Refer to [5.7 Alarm Test Call](#).

IMPORTANT: When the Pill Dispenser is delivered, make an alarm call test to check operation and to start the digital heartbeat. Refer to [5.7 Alarm Test Call](#).

7

Safety and Handling

The Pill Dispenser is an aid to help the user remember to take their medication. The dispenser is not suitable for everyone and assessment is very important. Legrand Care do not accept responsibility for faults or other consequences arising from any misuse of the dispenser.

It is the user's responsibility to ensure that medication is taken as prescribed.

- Read the manual before use.
- Only connect with other devices, chargers and power adapters that are approved by Legrand Care.
- Be careful with the device. Do not hit, shake, twist or let it fall.

7.1 Cleaning Pill Dispenser

- Use a weak soap solution or a surface disinfection and a moist cloth to clean the device.
- Do not use strong chemicals, grease and other harsh material.
- After cleaning, do an operational test, see [5.1 Self Test Function](#).

7.2 Discard

Do not put the used device and/or batteries in the household waste. If you put the device and/or batteries in the household waste, they can cause damage to the environment. To discard and recycle the device and/or batteries, refer to local regulations or your Legrand Care distributor.

8

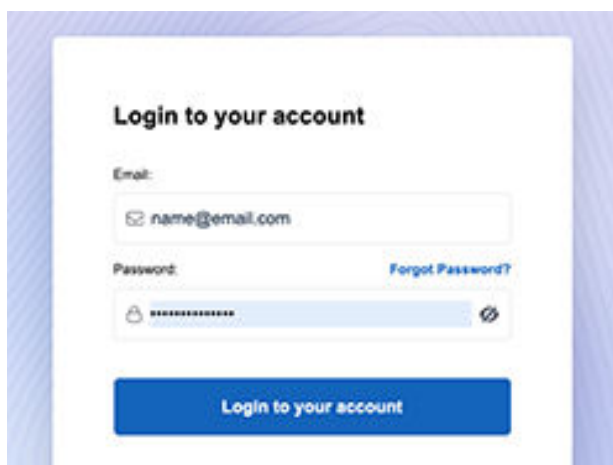
Technical Data

Measures (LxWxH)	190 x 180 x 56mm
Weight	390g excl. batteries
Operating temperature	Between +5 °C and +40 °C
Compartment volume	11 ml per compartment
Power supply	6V - 4 x AA Alkaline
Battery lifetime	> 1 year
Radio protocol	AALRP
Frequency _{RF} (EU/UK)	869.2125 MHz Class 1.5
Transmit power	10 mW (EIRP)

Appendix A. Connect Pill Dispenser to Telecare Receivers

1 Register the Pill Dispenser on a Reach IP

Register the unit via Pulse CMP:

- 1 Login to Pulse CMP.
- 2 Go to the ALARM UNIT view.
- 3 Enter the serial number of the Reach IP.
- 4 Go to *Accessories*.
- 5 Enter the Radio ID code, Location and Equipment type.
- 6 Press Save.
- 7 Press Synchronize.
- 8 Do an operational test.

Alternatively, register the unit via Reach IP:

- 1 Set Reach IP to normal operating mode.


- 2 Release the fixing screw and remove the back cover of the Reach IP.
- 3 Press the Power button with a **circle symbol** to enter Control mode.
- 4 Press and hold the button with a **Yellow square symbol** until the unit says "add radio device".
- 5 Release the button.
- 6 Press the button with the **the Red circle symbol** to confirm.
- 7 Do an alarm test call from the Pill Dispenser.
- 8 The Reach IP unit will make a loud beep sound signal and say "range test mode" if the device is new or not already paired.
- 9 Press the **alarm button** to confirm, or
 - press the **Yellow square symbol** to step to the next function, or
- 10 Press the **Red circle symbol** to exit the Control mode.

**IMPORTANT**

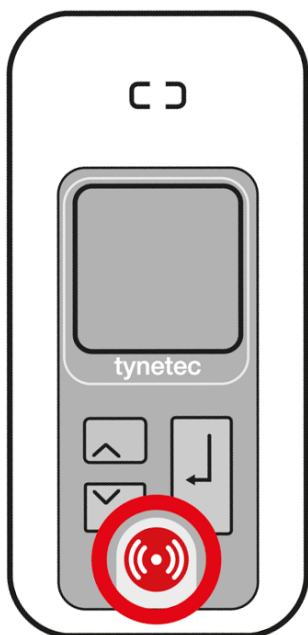Do an alarm test call within the first 9 hours to start the digital heartbeat.

2 Pair the Pill Dispenser with Carer Response

Refer to the *Carer Response User Manual* for full details of the functions and how to navigate through the menus.

- 1 Go to the Main menu.

- 2 Select **Programming**.
- 3 Select **Add transmitter** from the programming menu.
- 4 The display will tell you to *Activate the transmitter*.
- 5 Do an alarm test call from the Pill Dispenser.
- 6 The Carer Response makes a sound signal, and the display shows the **Add transmitter** menu with a field that says **No ID**.
- 7 Press \leftarrow then \blacktriangledown to select **Room**, then press \leftarrow .
- 8 Press \blacktriangledown to select the room number field **0000**.
- 9 Press \leftarrow to select the first digit, press \leftarrow to move between the digits, and press \blacktriangle or \blacktriangledown to change each digit.
- 10 Press \leftarrow when the correct room number is displayed.
- 11 Press \blacktriangledown to select the alarm type field **None**.
- 12 Press \leftarrow , then \blacktriangledown to select **Assistance**, then press \leftarrow .
- 13 Press \blacktriangledown to select **Save** then press \leftarrow .
- 14 The display shows **Radio ID saved to memory ...**
- 15 Go to the Main menu and select **Quit**.
- 16 Do an alarm test call.

3 Register the device in Touchsafe Pro

Register the Pill Dispenser through the *Master Touchsafe Pro Nursecall Panel*. V4.01 or later is required.

- 1 Login with the Manager or Engineer password.
- 2 Go to the Main menu.
- 3 Select **Telecare**, and this view shows:

- 4 Tap on the field **Add Telecare Device**.
- 5 Use the keypad on the screen to enter the 10-digit *Radio ID code* of the Pill Dispenser.
- 6 Tap on the field **Filter box**, and enter the *zone* where you want to add the Pill Dispenser.
- 7 Tap on the field **Room ▼** and select the *Callpoint ID* from the list that will be used to cancel the Telecare alarm.
- 8 Press **Save** on the keypad.
- 9 Press **Close** to exit.
- 10 Do an alarm test call.

4 Register the device in Advent XT2

Radio device programming can only be performed from the Advent XT2 Manager's Panel designated as the Controller.

- 1 Login with the Manager or Engineer password.
- 2 Press the Programming button.
- 3 Select the Program Radio Devices tab on the bottom of the screen.
- 4 Enter the Radio ID code: a 10-digit code from the label on the device.
- 5 Enter the **Location**: select where the device is located from the drop-down list. For example "Living Room". Select "no location" if it is a portable device.
- 6 Press the Send button to update the Advent XT2 system.
- 7 Do an alarm test call.

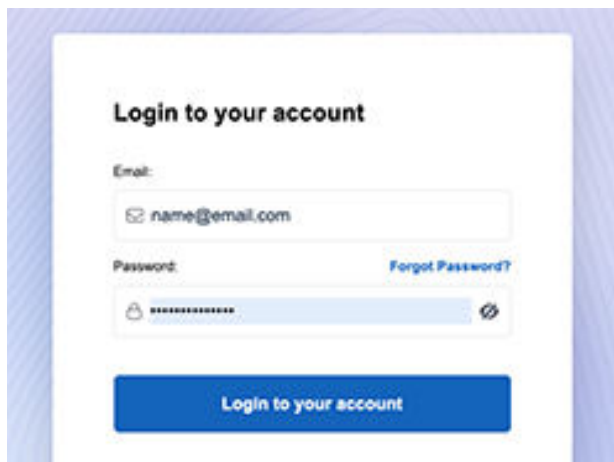
5 Register the Pill Dispenser on a NOVO

Make sure that the NOVO is on, and that the AC adapter is connected.

There are 2 ways to pair the Pill Dispenser with NOVO:


- 1 Through Pulse CMP
- 2 Through the Service Mode in the NOVO

5.1 Through Pulse CMP

- 1 Login to Pulse CMP.
- 2 Go to the **Carephone** view.
- 3 Enter the serial number of the NOVO.
- 4 Make a search to locate the NOVO.
- 5 Select **Accessories**.
- 6 Select **New**.
- 7 Enter the 10-digit *Radio ID code* of the Pill Dispenser. The code is located on the label of the Pill Dispenser.
- 8 Press **Save**.
NOTE: It can take up to 30 minutes before the accessory has been added.
- 9 Do an alarm test call from the Pill Dispenser.

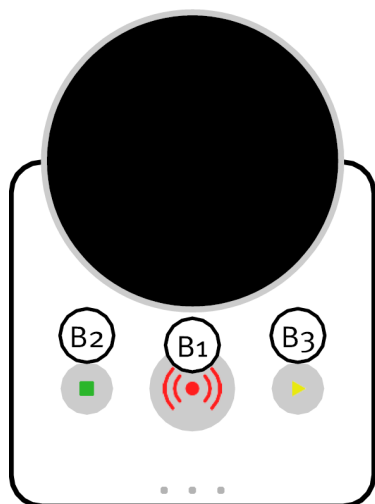5.2 Through the Service Mode in the NOVO

- 1 Press the Power switch briefly to enter Service mode.
The 3 buttons on the NOVO light up.
- 2 Press and hold button B3 until NOVO says "Add transmitter".
Button B1 lights up and Button B3 flashes.
- 3 Do an alarm test call from the Pill Dispenser.
The NOVO makes a loud sound signal and says "range test mode". The pairing is confirmed.
- 4 Press button B2 to exit.
- 5 Do an alarm test call from the Pill Dispenser to confirm the operation in the Pulse CMP Alarm log.

Legrand AB
Teknikvägen 5
246 43 Löddeköpinge
Sweden
+46 46 707065
www.legrandcare.com
infosweden.legrandcare@legrand.com

©2026 Legrand AB

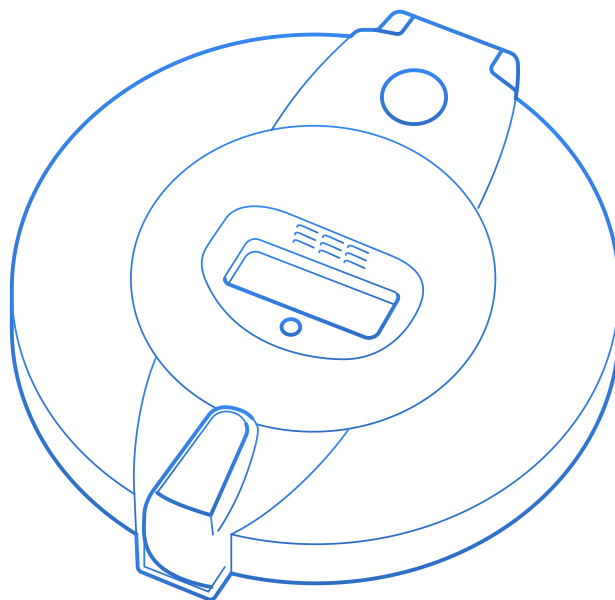
All rights reserved. Circle font is used with attribution to www.photoshopsupply.com

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